# sealart

## Root Canal Sealer Kök Kanal Dolgu Pati

### SEALART IS A RESIN-BASED ROOT CANAL SEALER OF THE HIGHEST QUALITY

#### Direction for using T-Endo SEALART

T-Endo SEALART is an epoxy resin based root canal sealer which is a paste-paste type of dual syringe. It has an outstanding chemical and physical property including extremely excellent sealing property and biocompatibility

SEALART complies biocompatibility with the requirements of ISO6876 for dental root canal sealing materials

#### Composition

- Base  
Epoxy oligomer resin  
Ethylene glycol salicylate  
Calcium phosphate  
Bismuth subcarbonate  
Zirconium oxide
- Catalyst  
Poly aminobenzoate  
Triethanolamine  
Calcium phosphate  
Bismuth subcarbonate  
Zirconium oxide  
Calcium oxide

#### Characteristics

When using SEALART in combination with gutta-percha points, this material shows the following properties:

1. Easy to mix by paste:paste
2. Hermetic sealing ability
3. Nonstaining to teeth
4. Insoluble in tissue fluids
5. Excellent biocompatibility
6. Good radiopacity
7. Working time of 35 minutes at 23°C(73°F)
8. Setting time of 45 minutes at 37°C(99°F)

#### Indication

Permanent obturation of root canals in combination with obturation points

#### Contraindications

Known hypersensitivity against epoxy resins or other components, respectively, of the root canal filling material

#### Mixing

Mix automatically pushed out volume units (2:1 wt. Ratio) of Base and Catalyst from the dual syringe on a mixing pad using spatula for 15-20 seconds or until creamily homogeneous consistency

#### Application

Canal walls must be dry. SEALART should be carried to the canal with either lentulo spiral or obturation points (gutta-percha, silver or titanium root canal points).

To avoid the formation of air bubbles in the material and overfilling of the canal, Advance the lentulo spiral slowly to the apex running at very low speed. Withdraw lentulo very slowly still running at low speed. Dip the disinfected and dry master point into SEALART and with pumping motions slowly push it into the canals.

SEALART is an endodontic product for every dental practice. It can be used for all established and more recent root canal sealing techniques. It is suitable for the single-point method, thermoplasticize method and all condensation techniques

#### Removal of root canal filling

If SEALART is used in combination with gutta-percha points, the root canal filling can be removed using conventional techniques for the removal of gutta-percha

#### Warnings

SEALART contains epoxy resins or amines which may cause sensitization in susceptible persons

#### Cautions

Do not use SEALART in allergic persons For dental use only Keep away from children

#### Side effects

Not known. In rare cases contact allergies are described

### INSTRUCTION FOR USE

#### Storage and handling

- Keep syringe cap tightly closed
- Store at 18°C~24°C(64°F~75°F)
- Store away from UV light

#### Shelf life

- 2 years from the manufacturing date

#### Components

- 13.5g Dual syringe SEALART (Base 9g, Catalyst 4.5g)
- Mixing pad
- Spatula
- Directions for using SEALART

#### Symbols

- Manufacturer
- Date of manufacture
- Batch Code
- Use By
- Consult instructions for use
- CE-Mark and identification number of notified body.
- Product conforms to the essential requirements of the Medical Device Directive 93/42/EEC
- Keep away from sunlight
- Temperature limitation
- Authorized representative in the EC

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**META BIOMED CO.LTD.**
EC REP **Meta Biomed Europe GmbH**